



Male Factor Pak Instructions for Use

1. Complete the patient information on the bag and save the bag for specimen transport.
2. Wash and dry hands and penis.
3. Remove the Male Factor Pak condom from bag and place against top of erect penis. To prevent trapping air, hold reservoir tip while unrolling condom onto penis. Proceed with your physician advised method of achieving ejaculation.
4. After ejaculation has occurred and erection has ceased, remove the Male Factor Pak condom being certain not to lose any of the specimen.
5. Use the enclosed twist tie to tie off the open end of the Male Factor Pak condom.
6. Place the Male Factor Pak condom into the bag and return it to your physician or laboratory as soon as possible, preferably within one hour of collection. Specimen should be maintained at or near body temperature during transport.